
Instructions

- 1** Secure your bur to the float using a “T” Wrench and Disk Wrench or Disk Pliers.



- 2** Attach cable securely to float. If using a key drive cable and adaptor, follow the instructions included with your cable.

- 3** Apply grease to the grease port located on the side of the float head.



For Instructional Videos and More Visit
www.EquineDentalTools.com

www.EquineDentalTools.com



For More Tips and Information Visit
www.EquineDentalTools.com


JERRYs

Equine Dental Tools

Address:
9225 Rocky Canyon Road
Atascadero, CA 93422

Telephone:
805-460-7276

Email:
jerry@jerryseqdentistry.com

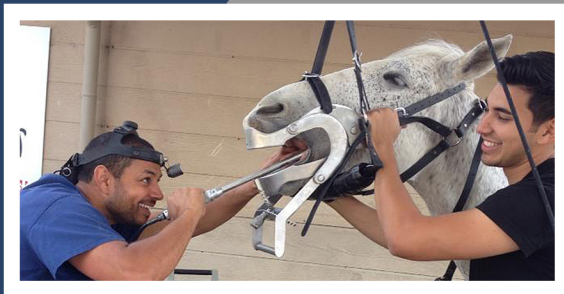

Suggested Maintenance

1 As you work, cool and clean your float in a bucket with water and Chlorhexidine. This will prolong the life of the tool.

Be sure to grease your float after every few horses, and again at the end of your work. This will help avoid rust buildup.

2

If your tool is need of repair or maintenance, send it in to our top-notch repair specialist!

Warranty Information

Our tools are thoroughly inspected and tested in our machine shop. All new tools sold by Jerry's Equine Dental Tools are warranted to be free of defects from workmanship and materials for the period of 6 MONTHS from the date of the original purchase.

This warranty does not apply where:

- Repairs have been made or attempted by others
- Repairs are required because of normal wear and tear
- The tool has been abused, misused or improperly maintained
- Alterations have been made to the tool

IN NO EVENT SHALL JERRY'S BE LIABLE FOR ANY INDIRECT, INCIDENTAL, PUNITIVE OR CONSEQUENTIAL DAMAGES ARISING FROM THE SALE OR USE OF THE PRODUCTS SOLD HEREUNDER