
Instructions

- 1** Make sure the battery pack is completely charged before the first use. Use dials on top and back of headlight to adjust size

- 2** Connect battery cable to headlight unit.



- 3** Press button on light to turn on and change brightness. Hold down button to turn light off.



For Instructional Videos and More Visit
www.EquineDentalTools.com

www.EquineDentalTools.com



Jerry's Bright
Headlight

For More Tips and Information Visit
www.EquineDentalTools.com

JERRY'S

Equine Dental Tools

Address:
9225 Rocky Canyon Road
Atascadero, CA 93422

Telephone:
805-460-7276

Email:
jerry@jerryseqdentistry.com



Suggested Maintenance

1 To clean the headlight, use a warm, damp cloth. Never submerge the unit in water.

If the lamp is warm to the touch, switch to the lowest setting, or let the unit rest for 5 minutes.

2

If your tool is need of repair or maintenance, send it in to our top-notch repair specialist!

Warranty Information

Our tools are thoroughly inspected and tested in our machine shop. All new tools sold by Jerry's Equine Dental Tools are warranted to be free of defects from workmanship and materials for the period of 6 MONTHS from the date of the original purchase.

This warranty does not apply where:

- Repairs have been made or attempted by others
- Repairs are required because of normal wear and tear
- The tool has been abused, misused or improperly maintained
- Alterations have been made to the tool

IN NO EVENT SHALL JERRY'S BE LIABLE FOR ANY INDIRECT, INCIDENTAL, PUNITIVE OR CONSEQUENTIAL DAMAGES ARISING FROM THE SALE OR USE OF THE PRODUCTS SOLD HEREUNDER